

## PM CHECK LIST

Sterilizer PM Kit (RPI Part #PCK222) for parts to fit the Validator® 8 Plus sterilizers

PART	RPI PART #	STERILIZER PM TIP
□ Door Gasket	PCG065	Remove the old door gasket. Clean the gasket groove and chamber face ring surface. For easy install of the new gasket, use the "star" or "North-South-East-West" method (ie. begin installing the gasket at the top of the door, then the bottom, then the left side, then the right side, and then eveything in between making sure that the gasket fits flush without bumps or gaps).
☐ Door Spacer	PCS067	Take CAUTION when removing or installing the door spacer – the doors tabs can be sharp. To install door spacer, place the spacer on and over the door tabs. The spacer MUST lie flat against the door.
Filter Fill Line (Chamber)	PCF009	Remove all trays from the chamber. Remove the chamber filter. Clean chamber. Install new filter making sure that the it is placed at the bottom and center of the chamber.
Reservoir Filter Kit	PCK167	Remove the sterilizer cover, drain and clean the reservoir if necessary when removing the old reservoir filter. Be sure to install the new o-ring when installing the new filter. Refill with distilled water only.
Sterilizer Sterilizer	-	After replacing the PM parts:  • If you have not yet checked and cleaned the Chamber and Reservoir, do so at this time. (Note: Debris is the number one reason for Solenoid Valve failure).  • Refill reservoir with distilled water only.  • Conduct a temperature check using a Max Register Thermometer (RPI Part #RPT113).  • Conduct a pressure check to identify any leaks that may need to be addressed.



## replacement parts industries, inc.

CALL (800) 221-9723 or (818) 882-8611 • FAX (818) 882-7028 E-MAIL order@rpiparts.com • WEBSITE www.rpiparts.com